# Moocall

## CALVING SENSOR

## User Guide

Gebruikershandleiding / Guide de l'Utilisation /  
Guida dell'Utente / Benutzerhandbuch / Manual del usuario /  
Podręcznik użytkownika / Használati útmutató / Guia do usuário

If you have any questions, please don't hesitate to contact us:

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**ENGLISH**

# Thank you for choosing Moocall

We here at Moocall are proud of our products and we hope that you will be as satisfied with your Moocall calving sensor as we are.

We look forward to many safe and successful calvings with you!

If you ever need our help, email [help@moocall.com](mailto:help@moocall.com), chat to us via our live chat on [www.moocall.com](http://www.moocall.com) or call us on the numbers below.

Very best regards,

Team Moocall

-  [www.facebook.com/moocall](https://www.facebook.com/moocall)
-  [www.instagram.com/moocallofficial](https://www.instagram.com/moocallofficial)
-  [@moocallsensors](https://twitter.com/moocallsensors)
-  [@moocallsensors](https://www.snapchat.com/add/moocallsensors)

Call us on the numbers below:

Telephone:

 01 969 6038

 0203 627 1126

 1-800-657-4291

International: +353 1 969 6038

## Step 1: Activate your Moocall calving sensor

In order to use your new Moocall calving sensor you must first activate it online.

Go to [www.moocall.com/register](http://www.moocall.com/register) to complete activation.

If you do not have access to the internet, call us to activate your sensor.

01 969 6038

0203 627 1126


1-800-657-4291

International: +353 1 969 6038

If the sensor has not been activated, it will turn itself off immediately after being turned on.

## Step 2: Adjusting the rubber liner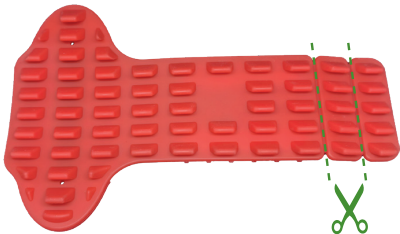


The rubber liner can be cut to size depending on the size of the tails on the cows in your herd. For thin tails, cut off both segments, for medium tails cut off one, and for thick tails don't cut at all.

If your herd has mixed tail thicknesses, contact Moocall to purchase additional rubbers.

## Step 3: Charging

To charge your Moocall calving sensor, flip away the rubber liner to expose the charge cap catch, remove the charge cap and insert charge cable as shown.

When charging is complete, replace the charge cap and fit the liner back around the charge cap.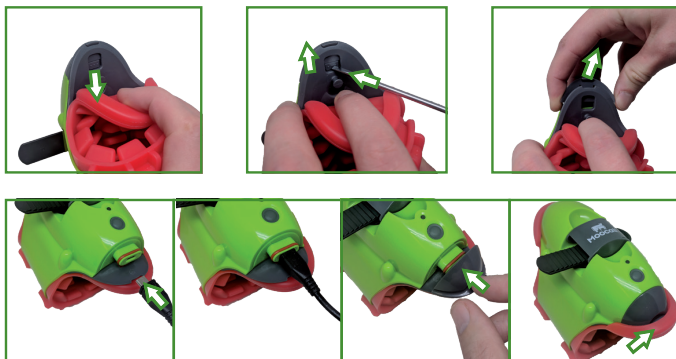


**IMPORTANT:** Ensure the charge cap is dry before placing it back on the sensor to be sure of a proper, waterproof seal.

DO NOT force the USB cable into the charging port of the sensor.


## Step 4: Turning on sensor

To turn on, press and hold the button for four seconds.

The light on the sensor will blink through the colours then settle on blue every 4-5 seconds, indicating that the sensor is on and active.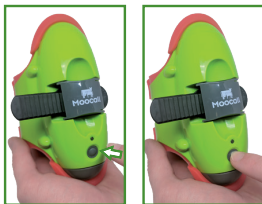

Hold the button for ten seconds to power off.

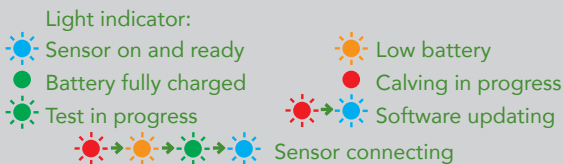The sensor will notify you when the battery level goes below 15%.

## Step 5: Testing sensor

Please perform this test in the area the sensor will be used to ensure network coverage is adequate for the sensor to function.

If you do not receive a message within ten minutes of performing this test, contact Moomall customer support.

Should the sensor never settle on blinking blue and turn itself off after it has been blinking multiple colours for a few minutes, you either need to activate your sensor or you may have a mobile network coverage issue in your area. Please contact Moomall.

Ensure that the sensor is on and that the blue light is blinking

Press the button firmly for half a second

Release and press the button again immediately

The sensor should start blinking green, indicating that the test is underway

If the green light doesn't start blinking on the second press, ensure that you're making solid contact with the button and try again.

You will receive an SMS on all numbers connected with the sensor, an email and a notification via the Moomall app if you have that installed.

Only press the button with your finger, do not use any tools as this may damage the button and the electronics inside.

## Step 6: Opening the clasp

Lift the paddle to pop open the Mocal door. Push paddle all the way to the left until it clicks open to loosen strap.

## Step 7: Placing the sensor on the tail


Place the sensor on the tail of a cow close to calving. The sensor should be placed on the tail opposite the vulva.

Insert the toothed strap into the ratchet.

Close the ratchet and click the door in place over it. Pull on the loose end of the toothed strap until the sensor is firm on the tail of the cow.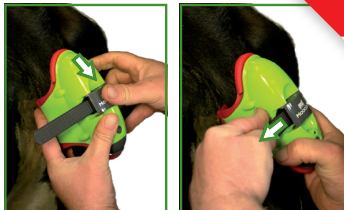

Ensure the sensor is firm on the tail by testing the fit. The sensor should be able to move side to side, but not downward on the tail. Adjust the tightness of the strap as needed until a proper fit has been achieved.

Video demonstration available  
[www.moocall.com/user-guide](http://www.moocall.com/user-guide)


### IMPORTANT! DO NOT OVER TIGHTEN!

If the sensor is placed on the tail too tight you will get false warnings and you may cause permanent, irreversible damage to the cow!

NOTE: Please ensure that your rubber liner has been adjusted to the size of your cow's tail. Using the wrong sized rubber can lead to discomfort for the animal and false warnings from the sensor.


## Step 8: Checking fit on the tail

Once the Moocall calving sensor is placed on the tail, please ensure the rubber liner isn't leaving the toothed strap rubbing on the tail or that the rubber itself is bunched up or overlaps the plastic housing to create pinch points.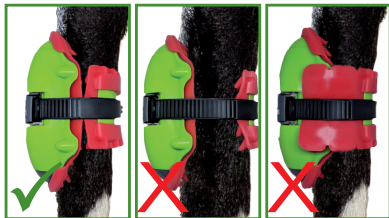


# Moocall SMS alerts


These are the SMS alerts the Moocall calving sensor will send to your phone.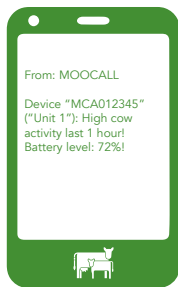


**First warning**

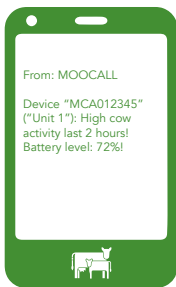**Second warning**

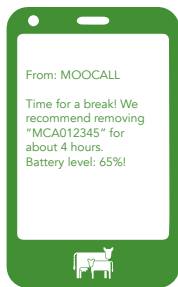**Break warning**

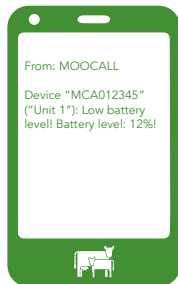**Battery warning**

**SMS Test**


**Off tail**

# MyMooCall dashboard

When you register your MooCall sensor you will choose your username and password for the MyMooCall dashboard. Here you can see the status of all your sensors in the left hand pane and all alerts sent from them in the center pane. You can manage your phone numbers in the right hand pane and your email alerts, time zone and language settings in the drop-down menu from your username in the top right.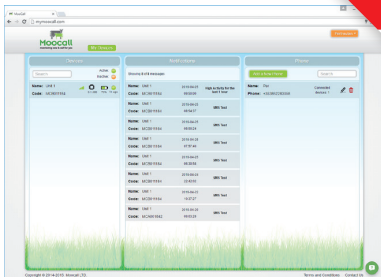

Access the MyMooCall dashboard via [www.mymooCall.com](http://www.mymooCall.com)

The MooCall Breedmanager app is available for Android and iOS devices. Search for "MooCall" in your app store and install. Log in to the app with the same username and password you use to access your MyMooCall dashboard.

The app gives you access to the same functions you have via the MyMooCall dashboard and much more: A herd management tool, ringtone notifications that makes your phone ring in addition to texts, Q&A forums and more. And it's all done over WiFi if the mobile signal around you is weak or absent, ensuring you'll never miss a notification.

## Step 9: Remove

Remove the sensor when it's safe to do so. Lift the paddle to open the Moocall door and press the paddle on the ratchet all the way to the left to loosen the strap.

## Step 10: Clean sensor

Clean gently with a dry brush or a damp cloth. Always clean the sensor after each calving.


**NOTE: DO NOT** submerge the sensor in water or other liquids!

**WARNING! DO NOT CLEAN SENSOR WITHOUT CHARGE CAP IN PLACE**

## Step 11: Resetting sensor

After a calving event, press the button for one second to reset. The light will change from solid red to blinking blue and the sensor is now ready to go on the next cow. The sensor will automatically reset after one hour if not reset manually. To turn the sensor off, press and hold the power button for ten seconds.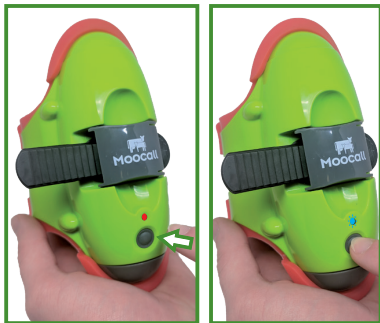

# Additional Sensor Information

## IMPORTANT!

The Moocall Sensor is designed to offer valuable assistance to farmers during the calving season. It is not guaranteed to predict every single calving event.

Reasonable levels of direct or camera supervision should be maintained during calving periods.

It is recommended that farmers use their own experience and knowledge of their herd to ensure best performance. For example, if the sensor is fitted to a very mature cow who delivers early or the cow does not experience the normal levels of distress or labour pains prior to calving the sensor may not send an alert to the farmers mobile phone.

Please only tighten the sensor to the point where it is secure and will not fall off the tail, if the sensor is overly tight it may cause discomfort to the animal and this may lead to false text alerts.

If cow has not calved within four days of attaching the sensor to the tail please remove unit from tail for 3-4 hours before reattaching again to avoid discomfort to the animal.

Moocall calving sensors are capable of detecting early labour distress even when this may not be visible to the farmer. Please regularly check and supervise your cow after notifications have been received.

Animal behaviour differs greatly from cow to cow. Moocall sensors are expected to predict a very high percentage of calvings (approx. 95%). Moocall Ltd do not take responsibility for animals lost or injured due to a short notice text or no text notice being given.

The Moocall calving sensor is designed for bovines only. Please do not use the sensor on other animals.

See [www.moocall.com/user-guide](http://www.moocall.com/user-guide) for up-to-date videos and information

# Battery Maintenance

To preserve the life of the battery and the longevity of the sensor, after your calving season, charge the battery before putting the sensor away.

**NOTE: FULLY CHARGE THE BATTERY BEFORE STORING SENSOR**

## Sensor Components

Charger

Plug AU/NZ

Plug US

Plug EU

Charge cable

Charge cap

Strap

Nut and screw

Adjustable ratchet

Rubber liner

Door

# My Moocall calving sensors

Write down your Moocall sensors' MC- and IMEI numbers here:

MC-number:

IMEI number:

---

---

---

---

## MyMoocall login

Write down your MyMoocall login details here:

Username:

---

Password:

---

[www.mymoocall.com](http://www.mymoocall.com)

# Contact Moocall

Our friendly and experienced customer service staff is always happy to answer all and any questions you may have about Moocall.

We can be contacted in many different ways:

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connecting you to your animals

MooCall is a registered trademark

You acknowledge and agree that the purchase of and/or use of the MooCall calving sensor (the "device") is governed by MooCall Limited's standard terms and conditions of supply which can be found at <http://mooCall.com/terms> (the T&Cs). The purchase of and/or use of the device by you shall constitute express acceptance by you of the T&Cs and an acknowledgement that you have read and understood the T&Cs.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC ID: R17GE865 / N7NHL8548

Troubleshooting:

Tel:

 01 969 6038

 0203 627 1126

 1-800-657-4291

International: +353 1 969 6038

Email: [help@mooCall.com](mailto:help@mooCall.com)